

How2Use Mix2Vial



1. Ensure product and diluent vial flip caps are removed and the stoppers are treated with an aseptic solution and allowed to dry before opening the **Mix2Vial** package.
-



2. Open the **Mix2Vial** package by peeling away the lid. Place the diluent vial on an even surface and hold the vial tight. Take the **Mix2Vial** together with the package and snap the blue end onto the diluent stopper.
-



3. Carefully remove the package from the **Mix2Vial** set. Make sure that you only pull up the package and not the **Mix2Vial** set.
-



4. With the product vial standing firmly on a surface, invert the diluent vial with set attached and snap the transparent adapter onto the product vial stopper. The diluent will automatically transfer into the product vial.
-

How2Use Mix2Vial cont'd



5. With the diluent and product vial still attached gently swirl the product vial to ensure the product is fully dissolved.
-



6. With one hand grasp the product-side of the **Mix2Vial** set and with the other hand grasp the diluent-side of the **Mix2Vial** set and unscrew the set into two pieces.
-



7. Draw air into an empty, sterile syringe. Keeping the product vial upright, screw the syringe onto the **Mix2Vial** set. Inject air into the product vial. With the syringe plunger still pressed down, turn the system upside down and draw the concentrate into the syringe by pulling the plunger back slowly.
-

8. Now that the concentrate has been transferred into the syringe, firmly grasp the barrel of the syringe (keeping the plunger facing down) and unscrew the **Mix2Vial** set from the syringe.